

# Premier $\alpha$

## Blood Glucose Monitoring System

### Specifications

- 0.4  $\mu$ L / 5 seconds
- 20-600 mg/dL (1.1-33.3 mmol/L)
- Hematocrit : 15-65%
- 1,000 test result memory
- 5-45°C
- Fresh capillary/venous whole blood

### FEATURES

- Enzyme : GDH-FAD
- No coding required
- 2.5 inches large display
- Strip ejection button
- Hypoglycemia indicator
- Strip expiration date indicator
- 1, 7, 14, 30, 90-day test averages (total, pre-meal, post-meal, fasting)
- Pre-meal, Post-meal, Fasting and No-mark flagging
- 4 alarms (PP2\* and 3-time set alarms)\*2 hours after meal
- Bluetooth®
- Data port



# Easy & Integrated Diabetes Care

## ONE-STEP OPERATION

- No coding
  - just insert the test strip and you are ready to test in one step.



Insert a test




Apply blood sample



Result in 5 seconds

## IMPROVED MEASUREMENT PERFORMANCE

- Advanced accuracy
  - 99.3% of results within  $\pm 15$  mg/dL (0.83mmol/L) or 15% of the reference results, meeting ISO 15197:2013 accuracy criteria.
- Wide hematocrit range
  - 15-65%
- Blood sample types
  - Capillary and zenous whole blood

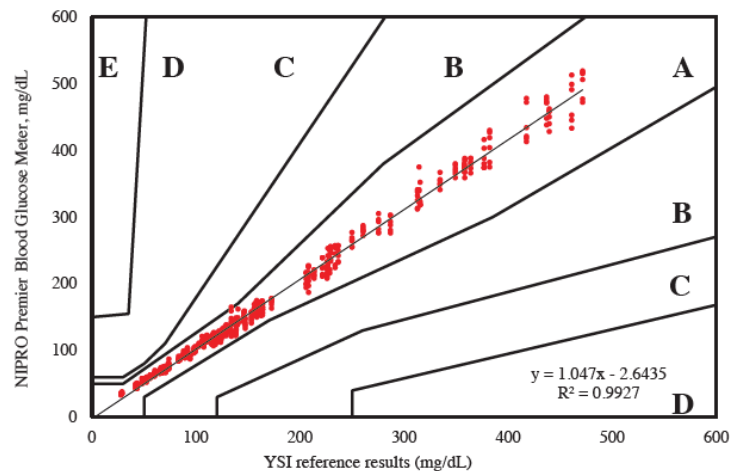

## USEFUL DESIGN



- 2.5 inches large, backlit display
- Strip ejection button (back)
- Confirmation window
- Pointed inlet

**NIPRO CORPORATION (Headquarters)**

3-9-3, Honjo-Nishi, Kita-ku, Osaka,  
531-8510, Japan  
Tel: +81 6 6372 2331 Fax: +81 6 6371 7422  
<https://www.nipro.com.jp/en/>

**NIPRO ASIA PTE LTD**

6 Temasek Boulevard  
#31-01/02 Suntec Tower Four  
Singapore 038986  
Tel: +65 6588 2281 Fax: +65 6588 2481  
Email: [niproasia@nipro.com.sg](mailto:niproasia@nipro.com.sg)  
<http://www.niproasia.com.sg/>

**NIPRO MALAYSIA SDN. BHD**

B-3-2 The Ascent Paradigm,  
No.1 Jalan SS7/26A, Kelana Jaya,  
47301 Petaling Jaya, Selangor, Malaysia  
Tel: +603 7622 1190/1191/1192  
Fax: +603 7622 180  
Email: [office@nipro.com.my](mailto:office@nipro.com.my)

**NIPRO JMI MEDICAL LTD**

Unique Heights, Level 11,  
117, Kazi Nazrul Islam Avenue, Ramna,  
Dhaka-1217, Bangladesh  
Tel: +880 2 55138723 Fax: +880 2 55138725  
Email: [info@jmigroup-bd.com](mailto:info@jmigroup-bd.com)

**NIPRO AUSTRALIA PTY LTD**

Suite 2. 02, 657 Pacific Highway,  
St. Leonards, NSW 2065, Australia  
Tel: +61 2 9437 9000 Fax: + 61 2 9437 0860  
Email: [support@niproaustralia.com.au](mailto:support@niproaustralia.com.au)  
<http://www.niproaustralia.com.au/>

**NIPRO MEDICAL PVT LTD**

24-CCA, DHA Phase 8, (Ex Park View),  
Lahore-54792, Pakistan  
Tel: +92 4237175802/4237175803 Fax: +92  
4237175804  
Email: [info@nipromedical.com.pk](mailto:info@nipromedical.com.pk)

**NIPRO SALES (THAILAND) CO., LTD.**

89/170 Nipro Building, Viphavadee-Rangsit Road,  
Talad-Bangkhen, Laksi, Bangkok, THAILAND 10210  
Tel : +66 2551 2000 Fax : +66 2551 2006  
Email : [sales@niprosales.com](mailto:sales@niprosales.com)

**NIPRO SALES VIETNAM COMPANY LIMITED**

Floor 15, Unit 15.06 & 15.07, Viettel Complex Build-  
ing 285 Cach Mang Thang 8, Ward 12, District 10,  
Ho Chi Minh city, Vietnam  
Tel : +84 908 260 405  
Email : [nhuy@niprosales.com](mailto:nhuy@niprosales.com)

**NIPRO SALES (THAILAND) CO., LTD. -  
MYANMAR REPRESENTATIVE OFFICE**

No. E 302, Manawhary Housing, Baho Road, Ahlone  
Township, Yangon, Myanmar  
Tel : +66 2551 2000 Fax : +66 2551 2006  
Email : [win@niprosales.com](mailto:win@niprosales.com)